

# Mask Assembly Instructions

## POM ELITE PACKAGE




1

1. Choose oral or nasal membrane port
  - Push finger from inside mask through the EZ tear Membrane
  - Fully separate connecting edges within membrane



2

2. Insert Medium Concentration adapter into O2 adapter port on mask
  - Make sure Adapter is fully seated and flush with inside surface of mask

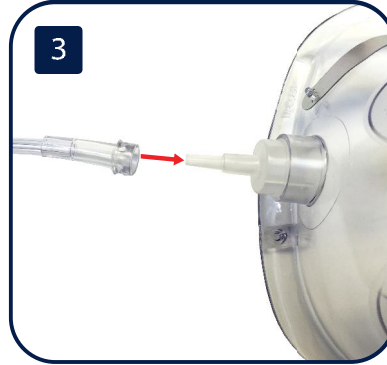

3

3. Securely attach O2 line to medium concentration adapter



4

4. Securely attach (Luer-Lok) ETCO2 sample line to ETCO2 adapter port on mask



5

5. Make sure O2 and ETCO2 sample lines are functioning properly



1

1. Choose oral or nasal membrane port
  - Push finger from inside mask through the EZ tear Membrane
  - Fully separate connecting edges within membrane



2

2. Insert High Concentration non-rebreather adapter into O2 adapter port on mask
  - Make sure non-rebreather adapter is fully seated and flush with inside surface of mask
  - Make sure one way valve on non-rebreather adapter is functioning properly



3

3. Securely attach O2 line to medium concentration adapter



4

4. Securely attach (Luer-Lok) ETCO2 sample line to ETCO2 adapter port on mask



5

5. Make sure O2 and ETCO2 sample lines are functioning properly