



MASK ASSEMBLY INSTRUCTIONS

ALL POM™ SERIES INCLUDE UNIVERSAL O₂ ADAPTER

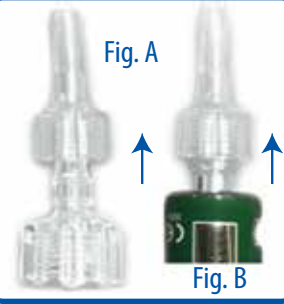


MEDIUM CONCENTRATION

POM 1003™ EZ SERIES

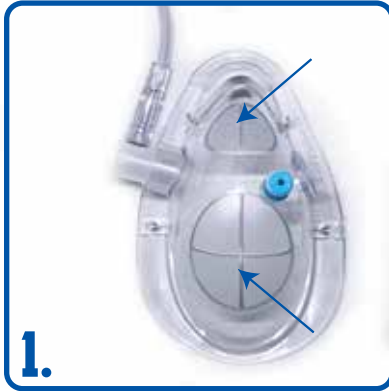

1. Choose oral or nasal membrane port, Then push finger from inside mask through the EZ tear membrane. Fully separate connecting edges within membrane.

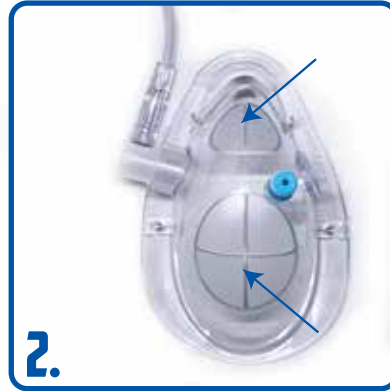

2. PRE-ASSEMBLED Universal O₂ line attached to medium concentration adapter



3. First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO₂ sample line to threaded EtCO₂ adapter port on mask.



4. Make sure Universal O₂ and EtCO₂ sample lines are functioning properly.

HIGH CONCENTRATION

POM 1001 LT™ SERIES



1. Choose oral or nasal membrane port, Then push finger from inside mask through the EZ tear membrane. Fully separate connecting edges within membrane.



2. Choose High or Medium Concentration. Insert Medium or High Concentration non-rebreather adapter into Universal O₂ port on mask. Make sure non-rebreather adapter is fully seated and flush with mask. Make sure one-way valve or non-rebreather adapter is working properly.



Securely Attach Universal O₂ line to either medium or high concentration adapter.



3. First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO₂ sample line to threaded EtCO₂ adapter port on mask.



4. Make sure Universal O₂ and EtCO₂ sample lines are functioning properly.